


Instructions For Use

Solid/Soft Clinical Waste Disposal Flatpack Carton

1

To open the cardboard container


Hold two corners of the flat packed box and pull them apart in opposite directions. Place your arm inside the container to help separate if needed.

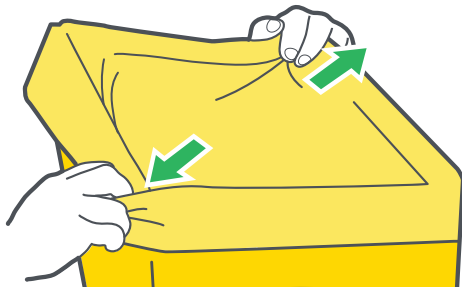

2

Lock the container open


Turn the carton over and push the sides until the automatic bottom locks open.

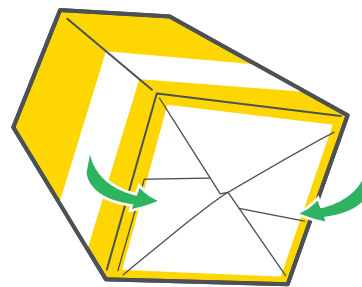

3

Fill in the label before and after use


This ensures a permanent audit trail. It is located on side B of the container.

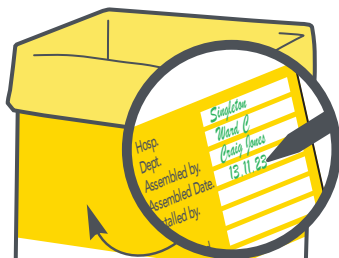

4

Place the container into a plinth


All cardboard containers should be placed into a plinth to prevent the base of the container getting damaged or wet.

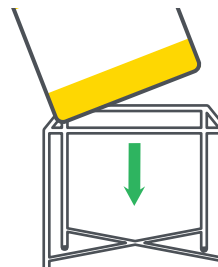

5

Secure the bag insert

Open the bag and place around the perimeter of the carton. Secure in place by folding the bag over each edge of the container.

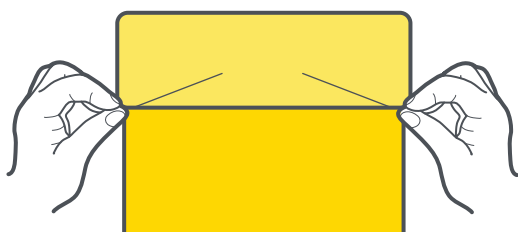

6

Do not overfill the container

Each cardboard container has a weight limit. Familiarise yourself with the weights of the containers and ensure you replace the container before it is overfilled.

