

Instructions For Use

Portable Range 0.2L/ 0.45L/ 0.6L


1 Pull out the safety handle

This allows one-handed use of the container in order to safely dispose of sharps and ensures the product can be carried safely.



2 Open flip lid to dispose sharps

Press down on the container top that is closest to the handle to flip the lid open.

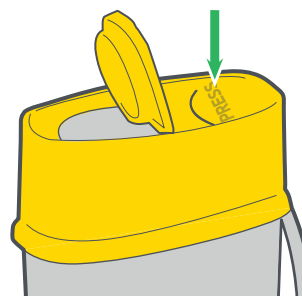


3 Temporary closure

Gently push the lid down until the aperture is covered. Temporary closure feature should always be engaged when the container is not in use.



4 Never fill above the line

Always replace your sharps container when full to avoid needle stick injuries.

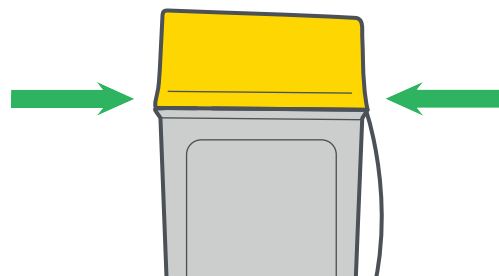

5 Fill in the label before and after use

This ensures a permanent audit trail.



6 Final closure

Final closure should be engaged when the container is filled to the fill line. With the container in temporary closure, press the lid down firmly until an audible click is heard. Fold the semi-circular tab down to lock the container.

